D/Sense® Crystal™

Dual-Action Crystal Precipitate Desensitizer Gel

INSTRUCTIONS FOR USE

EN – ENGLISH

PRODUCT DESCRIPTION: D/Sense Crystal is a one-step, dual action dentin desensitizer and cavity liner. It reacts with the smear layer to precipitate micro crystals of calcium oxalate and potassium nitrate. These crystals penetrate deeply into the tubules, and seal the entire dentinal surface with a continuous, acid-resistant complex (typically less than 3 microns thick). D/Sense Crystal may be used under crowns, bridges, and other restorative materials, or as a desensitizing treatment of cervical erosions. D/Sense Crystal is a patented solution of water, potassium binoxalate and nitric acid.

INTENDED USE: For the treatment of dentin sensitivity and as a cavity liner.


INDICATIONS: Dental restorations involving exposed dentinal tubules.

WARNING: Avoid contact with eyes. If contact occurs, flush immediately with water and seek medical attention. Always test flow of syringe and needle tip on a pad before intra-oral use. Federal law restricts this device for sale by or on the order of a dentist. There is a risk of infection if the single-use needle tip is used on more than one patient.

STORAGE CONDITIONS: Store at room temperature. Do not refrigerate. Cool temperatures may cause salts to precipitate out of solution. Warm material to body temperature before use on anesthetized patients.

DIRECTIONS FOR USE:

1. Remove cap from syringe and securely attach a SofNeedle foam tip.
2. To improve patient comfort, warm syringe to body temperature by holding under warm running water for 10 seconds.

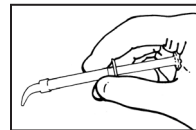3. **IMPORTANT!** For accurate control, hold syringe within the palm of the hand. Squeeze the plunger with the base of the thumb. DO NOT push the plunger with the tip of your thumb.

4. Check syringe flow. Test on a pad before intra-oral use. Squeeze syringe with a slow, steady pressure to wet foam tip.
5. Gently rub and saturate the dentin surface for 20 seconds. Be sure to saturate the area well. D/Sense Crystal may be applied to moist or dry dentin.
6. Use a gentle stream of air to dry the surface for 30 seconds. If not using compressed air, allow to air dry for 2 minutes. You will see a frosty white precipitate.
7. Above procedure may be repeated if needed. Dispose needle after use. Recap syringe.

CLINICAL APPLICATIONS:

- Use on teeth made sensitive from bleaching procedures.
- Use before cementation of temporary or permanent crowns and bridges. D/Sense Crystal is compatible with most types of temporary and permanent cements, including zinc phosphate, polycarboxylate, and glass ionomer.
- Use on exposed roots: D/Sense Crystal has low irritation potential, making it safe to use next to soft tissue.
- Use as a cavity liner under amalgam or composite restorations.
- Use on sensitive areas before and after scaling and root planning.
- Eugenol will not effect D/Sense Crystal. After cementation, apply directly to crown margins to seal any exposed dentin.

If any serious incident occurs, in relation to this device, report it to Centrix and the competent authority of the Member State of the user and/or patient.

Electronic instructions can be found here - <https://media.centrixdental.com/hubfs/ifu-sds/de/d-sense-crystal-ifu.pdf>

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Revised 2024/12 | ©2024 Centrix, Inc. | 012031 REV I