# FluoroDose®

5% Sodium Fluoride Varnish with Xylitol

## For printable Patient Take-Home Instructions visit: **centrixdental.com/takehome**

### ENGLISH INSTRUCTIONS FOR USE

FluoroDose® is a great-tasting, sodium fluoride varnish containing 50mg of sodium fluoride (equivalent to 22.6mg fluoride ion) per 1 ml. The convenient Lollitray™ single dose package allows easy mixing and application for uniform results.

**Directions for use:**

- Clean teeth and remove excess moisture from area to be treated.
- Grip the foil cover and slowly peel back, exposing the brush applicator and varnish well.
- Remove applicator brush and stir material in well.
- Paint a thin film of varnish onto treatment area.
- Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth.
- Dispose package and applicator after use.
- Patient may leave immediately after treatment. Patient should be advised to remain on a soft food diet and only drink cold liquids for two hours after treatment; and not to brush for a minimum of 4 to 6 hours.
- When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth.

**Contraindications:**

- Ulcerative gingivitis and stomatitis
- Known allergic reactions to fluoride
- Avoid ingestion during application
- Known allergies to colophony or similar materials

**Interactions:**

Do not apply when other fluoride treatments (gels or foams) are to be done on the same day. The use of fluoride tablets and rinses should be interrupted for several days after initial treatment.

**Adverse Reactions:**

Edematous swellings have been reported in rare instances, especially after application to extensive surfaces. Dyspnea, although extremely rare, has occurred in asthmatic patients. Patients with sensitive stomachs have reported nausea on occasion. Varnish may easily be removed through tooth brushing and rinsing.

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## SAFETY DATA SHEET (SDS)

In accordance with Hazard Communication Standard (HCS), 29CFR 1910.1200(g), 2012 revision

### SECTION I - PRODUCT / COMPANY IDENTIFICATION

Product Name: FluoroDose®

Synonyms: Sodium Fluoride Cavity Varnish

Description: Colophony desensitizing dental varnish, tooth shade A2, in various flavors

Company Name: Centrix Inc.

Address: 770 River Road

City, State, Zip: Shelton Ct, 06484

Emergency Telephone Number: 203-929-5582

Information Telephone Number: 800-235-5862

#### SECTION II – HAZARD IDENTIFICATION

Designation of Risk: Xi, irritant
☠
✖

Special Details of Risk (R-categories): 36-38

Detrimental effects/symptoms: Irritant in case of eye, skin and inhalation contact.

Harmful if swallowed. Contact with acids liberates very toxic gas. May cause sensitization by skin contact.

#### SECTION III - COMPOSITION / INFORMATION ON INGREDIENTS

Preparation: rosin-based varnish in ethanol carrier with 5% sodium fluoride

Hazardous Ingredients:

**Hazardous Components:**

Name	CAS-No.	Content	Symbol	R-Categories
Rosin	8050-09-7	50-70%		R43 EINECS: 232-475-7
Ethanol	64-17-5	10-30%		[F] R11 EINECS: 200-578-6
Sodium Fluoride	7681-49-4	1-10%	Xi	[T] R25; [-] R32;
			Xi	R36/38 EINECS: 231-667-8

Exact % content withheld due to trade secret restrictions.

#### SECTION IV - FIRST AID MEASURES

**Symptoms Action**

Skin contact: There may be mild irritation at the site of contact. Wash immediately with plenty of soap and water.

Eye contact: There may be irritation and redness. Bathe the eye with running water for 15 minutes.

Ingestion: There may be irritation of the throat. Wash out mouth with water.

Inhalation: No symptoms. Consult a doctor.

#### SECTION V - FIRE-FIGHTING MEASURES

Extinguishing media: Suitable extinguishing media for the surrounding fire should be used.

Use water spray to cool containers.

Exposure hazards: In combustion emits toxic fumes.

Protection of fire-fighters: Wear self-contained breathing apparatus.

Wear protective clothing to prevent contact with skin and eyes.

#### SECTION VI - ACCIDENTAL RELEASE MEASURES

Personal Precautions: Avoid contact of skin or eyes. Wash hands with plenty of water.

Environmental Precautions: Do not let the material get in contact with sewerage systems.

Dispose in accordance with all federal, state, and local regulations.

Methods for Cleaning Up: Take up mechanically, use common absorbents.

#### SECTION VII - HANDLING AND STORAGE

Storage conditions: Store in cool, well-ventilated area. Keep container tightly closed.

#### SECTION VIII - EXPOSURE CONTROLS / PERSONAL PROTECTION

Hazardous ingredients: ETHANOL

TWA: 1920 mg/m³

Respiratory protection: Respiratory protection not required.

Hand protection: Protective gloves.

Eye protection: Safety glasses. Ensure eye bath is on hand.

Skin protection: Protective clothing.

#### SECTION IX - PHYSICAL AND CHEMICAL PROPERTIES

State: Paste

Colour: Off-white

Odour: Characteristic odour

Evaporation rate: Moderate

Solubility in water: Slightly soluble

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### SECTION X - STABILITY AND REACTIVITY.

Stability: Stable under normal conditions.

Conditions to avoid: Heat.

Materials to avoid: Strong oxidizing agents. Strong acids.

Hazardous decomposition products: In combustion emits toxic fumes.

### SECTION XI - TOXICOLOGICAL INFORMATION.

Exposure Route: Refer to section 4 of SDS for routes of exposure and corresponding symptoms.

LD50:

ETHANOL SODIUM FLUORIDE

IVN RAT LD50 1440 mg/kg ORL MUS LD50 57 mg/kg

ORL MUS LD50 3450 mg/kg ORL RAT LD50 52 mg/kg

ORL RAT LD50 7060 mg/kg SCU RAT LD50 175 mg/kg

NTP Listing: not a known or potential carcinogen

### SECTION XII - ECOLOGICAL INFORMATION.

Mobility: Readily absorbed into soil.

Persistence and degradability: Biodegradable.

Bioaccumulative potential: No bioaccumulation potential.

Other adverse effects: Negligible ecotoxicity.

### SECTION XIII - DISPOSAL CONSIDERATIONS.

NB: The user's attention is drawn to the possible existence of regional or national regulations regarding disposal.

### SECTION XIV - TRANSPORT INFORMATION.

ADR / RID UN no: -

Shipping Name: NOT CLASSIFIED AS DANGEROUS IN THE MEANING OF TRANSPORT REGULATIONS.

IMDG / IMO UN no: -

IATA / ICAO UN no: -

### SECTION XV - REGULATORY INFORMATION

Hazard symbols: Harmful.

Risk phrases: R22: Harmful if swallowed.

R32: Contact with acids liberates very toxic gas.

R43: May cause sensitization by skin contact.

Safety phrases: S24: Avoid contact with skin.

S37: Wear suitable gloves.

PRECAUTIONARY PHRASES: Restricted to professional users.

Note: The regulatory information given above only indicates the principal regulations

specifically applicable to the product described in the safety data sheet. The user's attention is drawn to the possible existence of additional provisions which complete these regulations.

Refer to all applicable national, international and local regulations or provisions.

### SECTION XVI - OTHER INFORMATION

Risk phrases used in s.2: R43: May cause sensitization by skin contact.

R11: Highly flammable.
R25: Toxic if swallowed.

R32: Contact with acids liberates very toxic gas.

R36/38: Irritating to eyes and skin.

**CAUTION:** The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/ procedure will be exclusively under the user's responsibility.

	
<i>Making Dentistry Easier.<sup>SM</sup></i>	
©2021 Centrix, Inc.	Made in the USA
Date Revised: 3/2021	012051 REV M
<p><b>CENTRIX INCORPORATED</b> 770 River Road, Shelton, CT 06484 USA (203) 929-5582 (800) 235-5862 FAX (203) 929-6804 www.centrixdental.com</p>	
<p> Dental-Contact Vertriebs OHG, Im Splet 101, DE 26506 Norden</p>	
<p><b>EMERGO AUSTRALIA</b> Level 20, Tower II, Darling Park, 201 Sussex Street Sydney, NSW 2000, Australia</p>	