
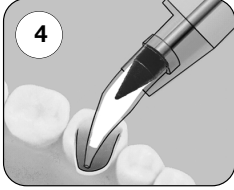
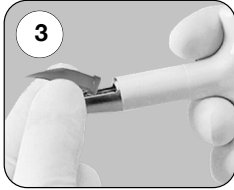
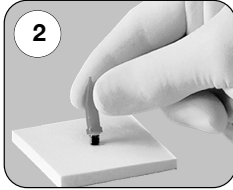
C-R® Syringe System

Precision Placement System for all Dental Materials

INSTRUCTIONS FOR USE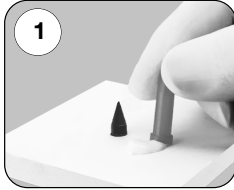

ENGLISH INSTRUCTIONS FOR USE

Centrix C-R® Syringe System: Centrix Composite-Resin (C-R) Syringes use high-leverage guns and traditional syringeaction for precision placement of dental materials. C-R Syringes are autoclavable.

C-R® Syringe Systems:

Mark I™, Mark II™ Snap-Fit™, Mark III™, S/Class®, E/Z™ Syringe, Snap-Fit™, Snub Nose Snap-Fit

Centrix C-R® Syringes are compatible with AccuDose Low Viscosity, High Viscosity, Medium Viscosity, NeedleTube and Ribbon Tube models; these tubes & plugs are single use only.

Directions for use:

1. Backload material into tube. Do not load more than half full!
2. Insert plug fully into tube.
3. Place tube into the syringe barrel.
4. Squeeze syringe with a slow, steady pressure.
5. Remove and dispose tube after use.

Sterilization Procedure for C-R® Syringes

After use, wipe off excess material and debris from syringe.

Clean in automated cleaning equipment (e.g., ultrasonic cleaner or washer-disinfector) to improve cleaning effectiveness. Manual cleaning should only be used if automated cleaning is not possible. Always follow the manufacturer's instructions.




Lift rear handle and remove black plunger. Remove metal spring from plunger.

- Automated Cleaning: Place all three parts (syringe, plunger, spring) into cleaning unit (or automated and clean according to the equipment directions. Remove and rinse thoroughly
- Manual Cleaning: Thoroughly clean by hand scrubbing with mild detergent and a soft bristle brush. Rinse thoroughly.

Dry: Remove excess water and air dry before sterilization.

Reassemble spring onto plunger making sure small spring hole is at back of plunger for proper reassembly (see photo showing correct method)

Sterilize: Bag, wrap in autoclavable wrap, seal, and arrange in gravity cycle sterilizer. Sterilize according to the sterilizer manufacturer's instructions (approx. 20 minutes at 250F/121C and 15psi).

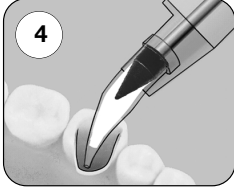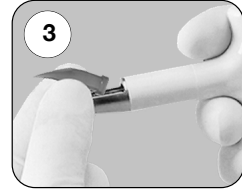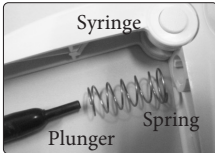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