# Access Crown®

## ENGLISH

**Instructions for use**

**[MD] UE Medical device**

**Access Crown® Cartridge** is supplied in 7 shades A1, A2, A3, A3.5, B1, C2 and BL.

**Product description:**  
**Access Crown®** is a fluorescent self-curing paste-paste system for the production of temporary crowns, bridges, inlays and onlays. **Access Crown®** consists of base paste and catalyst paste.

**Application:**  
**Access Crown®** in the cartridge:  
Insert **Access Crown®** cartridge into the **Centrix** Automix Gun dispenser or a compatible dispenser with a suitable piston. Due to technical reasons slight differences in the filling level between the cartridge chambers can occur. In this case, remove the cartridge cap and squeeze the material out until the material is delivered evenly from both chambers. Then insert a mixing tip and lock with a 90° turn clockwise. By pressing the lever the material is extruded and automatically mixed in the correct ratio. The warranted product characteristics can only be achieved when using the original **SuperMixer™** mixing tips as approved by **Centrix**. All the mixing tips for use with our respective products are available from **Centrix**. After use do not remove the mixing tip from the cartridge. Replace the used mixing tip with a new one just before the next application. When changing the mixing tips check the opening of the cartridge to ensure that the material can flow freely.

**Processing directions:**  
Level out any undercuts in the preparation using a non-eugenol cement. Prepare an impression in cases where no prefabricated crowns or similar restoratives are used. Prepare the impression by levelling undercuts and cutting drainage channels. If the wall thickness of the temporary restoration is too thin, enlarge the impression, especially removing the septums in the interdental embrasures. Then clean and air dry the impression.  
Apply **Access Crown®** directly into the impression. Do not overfill.  
**Access Crown®** attains an elastic phase after 1 - 1.5 min from beginning of mixing (approx. 0.5 - 1 min being in the mouth).  
The temporary should be removed from the mouth within 1.5 min after mixing, i.e. during the elastic phase.  
The setting process can additionally be monitored by observing the excess material in the mouth.  
The setting time of non-used material (in mixing tips) cannot be compared with the setting time of the material in the mouth.  
At a room temperature of 23 °C and 50 % humidity working times are as follows:

min <span> </span> : sec 0 <span> </span> : 00	min <span> </span> : sec 0 <span> </span> : 30	min <span> </span> : sec 1 <span> </span> : 00	min <span> </span> : sec 1 <span> </span> : 30	min <span> </span> : sec 4 <span> </span> : 00
Einsetzen in den Mund	Abbindung im Mund	Entnahme	vollständige Abbindung	Bearbeitung
Application in the mouth	Setting in the mouth	Removal	Complete setting	Finishing

Slight differences may result from different room temperatures.  
After approx. 2 - 3 min take the temporary out of the impression and wipe dry (if necessary, use a solvent, e.g. ethyl alcohol). This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. The surface of the temporary restoration should appear silky-matt.  
Carefully trim any excess material and roughen areas near the gingiva. Cement the temporary restoration using an eugenol-free temporary cement.

**Information, precautionary measures:**  
- **Access Crown®** contains methacrylates, amines, terpenes, benzoyl-peroxide, BHT. **Access Crown®** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.  
- Before making the temporary, the prepared cores can be insulated for desensitisation (fluoride varnishes, core varnishes, etc).  
- In order to avoid fractures or injuries from debris, the temporary should not be subjected to excessive loads (molar area).  
- Linings and cores built up with resin-based materials (composites) should be sufficiently insulated (e.g. glycerine) to avoid removal with the temporary.  
- Inadequate hygiene or intensive use of certain mouthrinses may lead to discolouration.  
- Finishing should only be started after complete setting of the material (approx. 4 min after mixing).  
- Restoration of defective temporaries:  
If the temporary should break despite the extreme stability of **Access Crown®**, the following procedures are recommended:
- **Air bubbles:** fill with **Access Crown®**.
- **In case of fracture immediately after the manufacturing of the temporary:** re-connect the fracture with freshly mixed **Access Crown®**.
- **Restoration of already used temporaries:** Roughen the fracture with a bur and place undercuts. Apply bond on the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Reassemble the fractured pieces and cure according to the instructions for use.

**Storage:**  
Store at temperatures of 4 °C to 23 °C. Do not use after expiry date. Protect from bright light, i.e. store in closed packages, drawers, etc.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/ or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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