Calciject®

Calcium Hydroxide Paste for Root Canal Therapy

EN – ENGLISH

INTENDED USE: Used as an intracanal medicament during root canal treatment, a direct pulp capping agent where the dental pulp is exposed due to trauma or decay, or to encourage the formation of a hard tissue barrier during apexification.

INDICATIONS: Its high pH and bacteriostatic action is of benefit when used in compromised root canals.

WARNING: There is a risk of infection if a single dose needle tube is used on more than one patient. Calcium hydroxide is corrosive and may dry the skin, causing irritation and burns. Avoid contact with skin and eyes. If accidental contact occurs, immediately flush with plenty of water. In case of eye contact, seek medical attention. Allergic or pre-existing skin, eye, and respiratory conditions may be aggravated by exposure to this device.

DIRECTIONS FOR USE:

- Remove Calciject cartridge from package.
- Pull off sheath cover and insert into Centrix C-R® syringe.
- Test first on pad. Squeeze with a slow, steady pressure.
- Bend needle, as needed, and inject into canal.
- After use, remove Calciject cartridge and dispose of properly.

If any serious incident occurs in relation to this device, report it to Centrix and the competent authority of the Member State of the user and/or patient.

 Electronic instructions can be found here: <https://www.centrixdental.com/media/pdf/ifu-sds/Calciject-Instructions.pdf>

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Revised 2024/07 | ©2024 Centrix, Inc. | 012018EU REV A