Connexio®

Self-Etch Bond

GB - ENGLISH

Instructions for use

[MD] UE Medical Device

Connexio® is a dual-curing, self-etching bond reinforced with nano-particles for creating a durable bond between tooth substance and light-, self- and dual-curing restoratives without marginal leakage. This all-in-one adhesive consists of 2 components, which eliminates the need for a separate etching step yet still achieves the same adhesive properties as total-etch bonds. **Connexio®** tolerates residual moisture and can be used universally with all lightcuring, self-curing and dual-curing composites, compomers and Ormocer®s.

Indications:

- Direct self-curing or dual-curing composite restorations and core build-ups
- Direct light-curing/composite/compomer/Ormocer®-based restorations
- Fixing of root pins/posts with dual-curing or self-curing composite cements
- Indirect restorations; when using dual-curing/self-curing composite cements for bonding of inlays, onlays, crowns and bridges

Application:

- Preventing contamination**
 - A dry working field is essential due to the subsequent application of the restorative. Avoid contamination with blood or saliva. Use of a rubber dam is recommended.

- Preparation**
 - Prepare the cavity according to the principles of the adhesive restoration technique. Rinse cavity thoroughly with water. Remove excess moisture with a faint air jet. Do not overdry dentine. The surface of the cavity should remain slightly moist. If the tooth has not been prepared (e.g. cervical defect) the surface must be thoroughly cleaned and the enamel margin roughened with a finishing diamond. Areas in close proximity of the pulp should be protected with a suitable calcium hydroxide-based liner.

- Mixing**
 - Connexio®** is dual-curing and therefore must be applied immediately after mixing. Mix 1 drop of liquid 1 and 1 drop of Liquid 2 on a mixing palette with a **Benda® Brush** for approximately 2 s (produces a self-etching adhesive).

- Connexio® application**
 - Apply the adhesive in a layer of medium thickness to the enamel/dentine and rub into the tooth surface for 20 s. Please note: The mixed bond is light-curing, thus should not be exposed to intensive ambient light. The operating light should be dimmed during application.

- Drying**
 - Dry the adhesive layer for at least 5 s with an air syringe.

- Polymerisation**
 - 6.1. Direct restoration**
 - Light-curing, self-curing and dual-curing materials**
 - The bonding layer must be polymerised with blue light (halogen or LED lights) for 10s when using light-curing, self-curing and dual-curing restoratives.

- 6.2. Indirect restoration**
 - Light-curing luting composites**
 - The bonding layer must be polymerised with blue light (halogen or LED lights) for 10s when using light-curing luting materials.

Self-curing or dual-curing luting materials
When using dual-curing or self-curing luting materials, **Connexio®** provides excellent adhesion without light-curing. **Connexio®** can also be light-cured for 10 s with blue light (halogen or LED lights) as an optional step.

Luting of root pins

- Massage the mixed **Connexio®** (Part A and B) into the occlusal surface of the core and into the root canal for 20 s with an applicator. Disperse solvent with an oil-free air-jet for 5 s. Remove excess material by dabbing it off with an absorbent strip of paper. Do not light-cure the bonding layer.
- Having prepared the root post in accordance with the manufacturer’s instructions, insert it into the root canal using a suitable luting material (see respective instructions for use). The aim is to obtain a slight excess of cement.
- Light-cure for at least 40 s to fix the root post and cure the **Connexio®** applied to the occlusal surface.
- Advice on luting Maryland Bridges**
 - For luting Maryland bridges, the enamel must either be roughened or conditioned with a phosphoric acid etching gel before applying **Connexio®**. Self-etching bonding materials are less effective on unprepared enamel. Overapplication of bonding material to enamel that has not been ground or bevelled can lead to margin discoloration. For optimal adhesion results with **Connexio®** the enamel should be prepared in advance in class III, IV, V cavities and for diastema closures.

Warnings and precautionary measuress:

- Do not fold blister before activation. Blisters with obvious damage such as creases or kinks should not be used.
- Do not damage either the blister foil or tab on either the top or the bottom side with fingernails.
- Do not use any type of instrument for activation. Only the thumb and index finger should be used for activation with the thumb placed exactly over the mark on the label and the index finger placed exactly on the opposite side of the package.
- Activation and subsequent puncturing of the foil with a **Benda® Brush** should not take place within close proximity to the eyes.
- Never simultaneously press the reservoir from both sides or press the liquid back and forth between the reservoirs.
- Phenolic substances (e.g. temporary materials containing eugenol) impair polymerisation and should therefore not be used in contact with **Connexio®**. **Connexio®** contains organic acids, Bis-GMA, HEMA, TMPMTA, campherchinon, amines (DABE), BHT, catalysts, fluoroethers and ethanol. **Connexio®** should not be used in case of known hypersensitivities (allergies) to any of these ingredients. Avoid contact with the oral mucosa. Contact of the oral mucosa with **Connexio®** may result in a whitish discoloration caused by the coagulation of proteins. This is a reversible occurrence that subsides within a few days. In case of contact with the eyes, rinse with plenty of clean water and seek medical advice.

Advice for storage and usage:

Tightly close **Connexio®** bottles after use. Store in an upright position to guarantee flow-back of the liquid. Store at temperatures between 4 °C - 23 °C. Allow **Connexio®** to reach room temperature before use. Do not expose to direct sunlight or the chairlight. Do not use **Connexio®** after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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