English INSTRUCTIONS FOR USE

For professional use only

Encore® Original
Self-cure composite resin for core build-ups

REF 310070 Encore White with Fluoride
REF 310075 Encore White without Fluoride
REF 310077 Encore Natural without Fluoride
REF 310076 Encore Natural with Fluoride

INSTRUCTIONS FOR USE

1) Prepare the tooth:
Remove all decay and old restorative material. Any eugenol present must be rinsed away with ethanol before proceeding.

2) Apply bonding agent.

3) Mix and apply Encore:
Using opposite ends of the mixing spatula, dispense equal amounts of Base and Catalyst pastes onto the mixing pad (note: for a longer working time, use a higher percentage of catalyst paste). Mix thoroughly for 10 to 15 seconds, then back-load into the NeedleTube™ syringe tip. Fill the tip no more than 1/2 full. Press filled tip over the green plug to seal it, and insert into a Centrix C-R® syringe gun. Bend needle if required. Inject Encore into the post hole and/or prepared area, squeezing the syringe with a slow, steady pressure. Keep the tip imbedded in the material as you inject, to reduce the chance of voids.

4) Finish the core
After material is fully set (typically 4 to 5 minutes after injection), Encore can be cut to the desired shape, using water-cooled diamonds or burs.

Storage: All kit components should be stored at room temperature, or within a range of 10°C-24°C (39°F - 73°F). Refrigeration may be used to extend shelf life. Return to room temperature before use.

Precautions:
Encore composite resin – Do not store or use in presence of eugenol. Do not use on persons with known sensitivities to composite resin restoratives.

SUGGESTED ACCESSORY:
REF 130650 Snap-Fit Syringe

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© 2015 Centrix, Inc.