FluoroDose®, a 5% sodium fluoride varnish, was applied to your teeth today. This treatment delivers fluoride to your teeth and holds it to your enamel. That feeling on your teeth means it is working. FluoroDose should be a regular part of your comprehensive oral hygiene program.

To receive the maximum benefits of today's treatment, the varnish should remain on your teeth for as long as possible, or at least for the next 4 to 6 hours. During this time, we recommend the following:

- **Drink only cold beverages and eat only cold, soft foods for the first two hours**
- **Avoid hot or alcoholic beverages**
- **Do not brush or floss your teeth for at least 4 hours**

After it dries, any excess varnish may appear "spotty" and may slide off the tooth surface. Any excess material that loosens may simply be spit out. Do not worry, this will not affect product performance as there is still a full layer of varnish on your teeth. FluoroDose may easily be removed by brushing your teeth and rinsing.

If you have any concerns, questions or problems, call our office immediately.

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