ENGLISH INSTRUCTIONS FOR USE FOR PROFESSIONAL USE ONLY

Encore® Original
Self-cure composite resin for core build-ups
REF 310070 Encore White with Fluoride
REF 310075 Encore White without Fluoride

INSTRUCTIONS FOR USE

1) Prepare the tooth:
Remove all decay and old restorative material. Any eugenol present must be rinsed away with ethanol before proceeding.

2) Apply bonding agent.

3) Mix and apply Encore:
Using opposite ends of the mixing spatula, dispense equal amounts of Base and Catalyst pastes onto the mixing pad (note: for a longer working time, use a higher percentage of catalyst paste). Mix thoroughly for 10 to 15 seconds, then back-load into the NeedleTube™ syringe tip. Fill the tip no more than 1/2 full. Press filled tip over the green plug to seal it, and insert into a Centrix C-R® syringe gun. Bend needle if required. Inject Encore into the post hole and/or prepared area, squeezing the syringe with a slow, steady pressure. Keep the tip imbedded in the material as you inject, to reduce the chance of voids.

4) Finish the core
After material is fully set (typically 4 to 5 minutes after injection), Encore can be cut to the desired shape, using water-cooled diamonds or burs.

Storage: All kit components should be stored at room temperature, or within a range of 10° C-24° C (50° F - 75° F). Refrigeration may be used to extend shelf life. Return to room temperature before use.

Precautions:
Encore composite resin – Do not store or use in presence of eugenol. Do not use on persons with known sensitivities to composite resin restoratives.

SUGGESTED ACCESSORY:
REF 130650 Snap-Fit Syringe

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SAFETY DATA SHEET (SDS)

In accordance with Hazard Communication Standard (HCS), 29CFR 1910.1200(g), 2012 revision

SECTION I - PRODUCT/ COMPANY IDENTIFICATION

Product Trade Name: ENCORE / ENCORE AF
Part (Item) Number: REF 310070, 310075, 310076, 310077, 310093, 310094
Product Name: Dental Core Build-Up material
Manufacturer: CENTRIX, INC. Address: 770 River Road
City, State, Zip: Shelton, CT 06484 U.S.A.
Emergency or Information Telephone Numbers: (203)929-5582, Toll-Free: (800)235-5862
(USA & Canada) Fax: (203)929-6804
Date Revised: 3/2015

SECTION II - HAZARDS IDENTIFICATION

Designation of Risk: Xi, irritant 
Special Details of Risk (R-categories): 36-38
Detrimental effects/symptoms: Irritant in case of eye, skin and inhalation contact. Harmful if swallowed. May cause sensitization by skin contact.

SECTION III - HAZARDOUS INGREDIENTS / IDENTITY INFORMATION

Hazardous Components:	%	CAS NO.	SYMBOL	R	OTHER LIMITS
Blend of multifunctional methacrylates	26 - 32	N/E	Xn/Xi	22/36/37/38	
Initiator	0.10 - 0.35	[94-36-0]	E/Xn	2/7/36/43	

Exact % content withheld due to trade secret restrictions.

SECTION IV - FIRST AID MEASURES

Skin: Immediately wash skin with soap and water. If irritation occurs seek medical advice.
Ingestion: If swallowed do not induce vomiting. Give large quantities of water and milk.
Seek medical attention. Inhalation: Remove to fresh air. If breathing becomes difficult, contact a physician.
Eyes: Flush with plenty of water. Seek immediate medical attention by an Ophthalmologist.

SECTION V - FIRE-FIGHTING MEASURES

The product is not considered to be a fire hazard. The information below is given for extinguishing a fire generated by other sources.
Suitable extinguishing media: Water, Chemical Foam, Carbon Dioxide, or Dry Chemical.
Forbidden extinguishing media: Not determined.
Special firefighting measures: None. Wear self-contained breathing apparatus.
Unusual fire and explosion hazards: Heat can cause polymerization with rapid release of energy.
Special protection equipment: Self Contained Breathing Apparatus if toxic fumes are generated.

SECTION VI - ACCIDENTAL RELEASE MEASURES

Personal precautions: Adopt some precautions of section VII.
Environmental precautions: Keep spilled material out of sewers.
Reclaiming methods: Absorb spills with inert material - keep spilled material out of sewers.

SECTION VII - HANDLING AND STORAGE

Handling: Avoid unnecessary exposure. Storage: Room temperature. Containers: Supplied by manufacturer. Incompatibilities: Avoid the contact with reducing and oxidizing agents, peroxides and amines. Other precautions: Use according to directions. Avoid contact with eyes, skin and soft tissue.

SECTION VIII - EXPOSURE CONTROLS / PERSONAL PROTECTION

General measures: Local Exhaust Ventilation: Good general ventilation should be sufficient to control airborne levels of vapors released by uncured material. Limits: TWA: Not determined; TLV: Not determined. Respiratory protection: Avoid prolonged or excessive breathing of vapors of uncured material. Eye protection: Safety glasses or goggles.
Skin protection: Respect good personal hygiene practice. Avoid unnecessary exposure.
Hands protection: Impervious gloves.
Other equipment: Lab coat.

SECTION IX - PHYSICAL AND CHEMICAL PROPERTIES

Appearance: Colored/uncolored paste Explosion limits: Not determined Odor: None
Water solubility: Insoluble Boiling point: Not determined
Evaporation rate: Not determined

Flash point: Not determined Other Data: Not available or none.

SECTION X - STABILITY AND REACTIVITY

Stability: Stable if stored as directed.
Conditions to avoid: Heat, light, aging and contamination.
Materials to avoid: Reducing and oxidizing agents, peroxides and amines.
Hazardous decomposition products: Oxides of Carbon.
Hazardous polymerization products: May occur.

SECTION XI - TOXICOLOGICAL INFORMATION

Ingestion: Uncured material may be harmful or fatal if swallowed.
Eye contact: May cause irritation and damage if not removed promptly.
Skin contact: Prolonged or repeated exposure to uncured material may cause irritation or skin rash. Inhalation: Prolonged or excessive inhalation may cause respiratory tract irritation. Prolonged exposure: Not applicable.

SECTION XII - ECOLOGICAL INFORMATION

There is no specific information on the product. Use according to good working practice. Avoid releasing the product into environment.

SECTION XIII - DISPOSAL CONSIDERATIONS

Unpolymerized (uncured) material may be hazardous waste.
Incinerate uncured material and dispose in accordance with local regulations.

SECTION XIV - TRANSPORT INFORMATION

Additional Information: Not dangerous cargo.
SECTION XV - Regulatory Information
CLASSIFICATION IN ACCORDANCE WITH EEC DIRECTIVES.
Hazard Classification Symbol: Xn
R22 Harmful if swallowed.
R36-38 Irritating to eyes, respiratory system and skin.
S36 Wear suitable protective clothing.
Ingredients determining the label: Blend of multifunctional methacrylate

SECTION XVI - OTHER INFORMATION

Handling of product exclusively by dental specialists according to instructions for use.
CAUTION: The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/procedure will be exclusively under the user's responsibility.

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EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex Street
Sydney, NSW 2000, Australia