# CENTRIX GingiTrac™ MiniMix® Gingival Retraction System

## ENGLISH

### DIRECTIONS FOR GINGIVAL RETRACTION

1. Prepare the tooth as normal.
2. If excessive bleeding, use a hemostatic/astringent agent.
3. Select appropriate size GingiCap.
4. Attach the mixing nozzle to the GingiTrac Retraction MiniMix.
5. Load MiniMix into Snub Nose Snap-Fit Syringe.
6. Express a “pea-sized” amount and discard. This is to ensure that the catalyst and base are being mixed thoroughly.
7. Fill the GingiCap with GingiTrac Retraction material. Do not allow filled GingiCap to remain at room temperature for more than 1 minute 30 seconds, otherwise material will harden.
8. Syringe the GingiTrac Retraction material directly around the tooth preparations’ margin using the included dispensing tip.
9. Place the filled GingiCap over the preparation.
10. Have patient close with firm but not excessive pressure. Wait 2 minutes, depending on the amount of bleeding or oozing, for complete set and retraction.
11. Remove the set GingiTrac. Check for complete retraction. If needed, repeat steps 7-10.
12. If necessary, gently rinse and dry the prepared teeth. You are now ready to take your standard impression.

**NOTES:** Do not use the GingiTrac™ as a final impression

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#### SECTION I - PRODUCT/ COMPANY IDENTIFICATION

Product Trade Name: GingiTrac™  
Common Name: Dental gingival retraction paste  
Part (Item) Number: REF 360068, 360069  
Manufacturer: CENTRIX INC  
Address: 770 RIVER ROAD, SHELTON, CT 06484  
Emergency Telephone Number: Toll-Free (800)235-5862, (203)929-5582

#### SECTION II - HAZARDS IDENTIFICATION

HAZARD CLASSIFICATION: non-hazardous, slight irritant for eyes

#### SECTION III - INGREDIENTS/IDENTITY INFORMATION

ADDITION SILICONE (VINYL TERMINATED POLYSILOXANE)  
Alum

#### SECTION IV - FIRST AID MEASURES

Eye Contact:	Flush immediately with plenty of water. Consult a physician if irritation persists
Skin Contact:	Wash with plenty of water and soap.
Ingestion:	Consult a physician.
Inhalation:	Not applicable.

#### SECTION V - FIRE-FIGHTING MEASURES

Suitable Extinguishing Media: Foam, dry powder, carbon dioxide, water spray  
Unusual fire and explosion hazards: None  
Specific intervention methods: Appropriate protective equipment must be worn in case of fire

#### SECTION VI - ACCIDENTAL RELEASE MEASURES

Recovery: Collect in a suitable container for disposal.  
Cleaning/Decontamination: Wipe up any affected area with absorbent material.  
Disposal: Dispose of material in accordance with local, state, and federal requirements.

#### SECTION VII - HANDLING AND STORAGE

**Handling**  
Technical Measures: Processing in closed systems, if possible superposed by inert gas (e.g. nitrogen)  
Take precautionary measures against electrostatic charging  
Suitable Materials: Stainless steel, aluminium, enamel, glass, polyethylene - test plastics before use

#### Storage

Handling:	Does not require specific technical measures.
Storage Containers:	Store in supplied packaging.
Storage Incompatibilities:	Strong oxidizing agents.
Storage Other precautions:	Store away from heat sources.

#### SECTION VIII - EXPOSURE CONTROLS / PERSONAL PROTECTION

Eye protection:	None required	Hands protection:	None required
Engineering controls:	None required	Other equipment:	None required

#### SECTION IX - PHYSICAL AND CHEMICAL PROPERTIES

Appearance:	Blue-green base/white catalyst, Viscous fluid
Vapor pressure:	< 5 mm Hg @ 25 C
Odor:	Mint (base)
pH:	Not applicable
Flash point:	> 450 °F (material will ash)
Boiling point:	>150 °C
Vapor pressure:	< 5 mm Hg @ 25 °C
Crystallization point:	Not applicable
Specific gravity:	>1
Solubility:	Insoluble in water
Volatile part by weight:	None volatile
n-octanol / water partition coefficient:	Not applicable


#### SECTION X - STABILITY AND REACTIVITY

Stability:	Stable
Materials to avoid:	No dangerous reactions known
Hazardous decomposition products:	None to our knowledge

#### SECTION XI - TOXICOLOGICAL INFORMATION

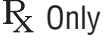Acute toxicity: Non-toxic  
Local effects: Eye irritant

## INSTRUCTIONS FOR USE

 Only      
10°C -24°C / 50°F -75°F

#### SECTION XII - ECOLOGICAL INFORMATION

Environmental fate: No data available  
Degradability: Non-biodegradable  
Ecotoxicity: Impact on aquatic environment: No data available. No affect anticipated.

#### SECTION XIII - DISPOSAL CONSIDERATIONS

Dispose in accordance with federal, state and local regulations

#### SECTION XIV - TRANSPORT INFORMATION

DOT- Not regulated

#### SECTION XV - REGULATORY INFORMATION

Federal - SARA title III section 313: This product does not contain any substance subject to reporting requirements at or above deminimis quantities.  
Federal - TSCA: All components are either listed or otherwise comply with requirements  
International - DSL (Canada): All ingredients are listed on the Canadian Domestic Substance List (DSL)  
International - WHMIS (Canada): Not a "controlled product" under the Canadian Workplace Hazardous Materials Information System- (WHMIS)

#### SECTION XVI - OTHER INFORMATION

None.

#### CAUTION

The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/procedure will be exclusively under the user's responsibility.

**centrix**

*Making Dentistry Easier.™*

	<b>CENTRIX INCORPORATED</b> 770 River Road, Shelton, CT 06484 USA (203) 929-5582 (800) 235-5862 FAX (203) 929-6804 www.centrixdental.com
 	Dental-Contact Vertriebs OHG, Im Spiet 101, DE 26506 Norden
<b>EMERGO AUSTRALIA</b>	Level 20, Tower II, Darling Park, 201 Sussex Street Sydney, NSW 2000, Australia