Glu/Sense™

Dentin Desensitizer

ENGLISH INSTRUCTIONS FOR USE

Glu/Sense™ Dentin Desensitizer

Glu/Sense dentin desensitizer works by stopping intratubular fluid shifts through polymerization with proteins within the tubules. No-drip formula aids in placement accuracy.

Indications for Use:

- Glu/Sense dentin desensitizer is used for the hypersensitive dentine for the following applications:
- To eliminate pain in exposed cervical areas not requiring restoration
- To alleviate or prevent dentinal sensitivity after preparation of teeth to receive indirect restorations

Cautions:

Glu/Sense dentin desensitizer should only be used by a dentist. Protect mucous membranes by using a rubber dam. Make sure the Glu/Sense dentin desensitizer only comes into contact with the area to be treated.

Active Ingredients:

35% H.E.M.A. (2-Hydroxyethyl Methacrylate Pentanediál), 5% Glutaraldehído

Warning:

Avoid contact with eyes, skin, and soft tissue. If contact occurs, flush immediately with water. Extended contact with tissue can result in redness or burning. Treat like any other chemical burn. If eye contact occurs, consult a physician immediately. Do not use on patients with known sensitivities to HEMA or glutaraldehído.

Storage Conditions: Store at room temperature, away from sunlight.

For Syringe Package Application:

- Remove syringe cap and twist on a SofNeedle™ applicator tip.
- Bend tip, as needed, to gain access to the application site.
- Before use, express a small amount of liquid onto a pad to insure that the material is flowing properly through the SofNeedle application tip.
- After use, remove SofNeedle and discard. Replace cap and disinfect syringe.

DIRECTIONS FOR USE:

Apply Glu/Sense desensitizer to the dentin of the tooth, using a gentle rubbing motion for 30 seconds. Dry and proceed to restoration when indicated. Avoid contact with soft tissue. If contact occurs, flush immediately with water. (See Warning section).

Tooth Preparation:

- Place pulp protection, when necessary, over areas very close to the pulp.
- Dentin does not need to be dry; a moist surface is acceptable.
- Removal of the dentinal smear layer is not necessary. However, if using a dentin-bonding agent that requires "total etch", remove the smear layer by etching first, and then apply Glu/Sense.
- Dry after applying Glu/Sense with stream of compressed air, followed by thorough rinse and suction.
- Apply each coat for 30 seconds, using a gentle rubbing motion.
- Glu/Sense is compatible with all restorative materials and may be used under composites, cements, glass ionomers and amalgam.

For Cervical Sensitivity (when no restorative material is to be placed):

- Clean surface area with pumice and rinse with water.
- If area is too sensitive to clean with pumice, skip this step and apply the smallest possible amount of Glu/Sense desensitizer in one or two coats. Apply each coat for 30 seconds, using a gentle rubbing motion.
- Dry thoroughly with gentle stream of compressed air until fluid film has disappeared and the surface is no longer shiny. Rinse thoroughly with water.
- Avoid contact with soft tissue. See warning above.
- In extreme cases of dentin hypersensitivity, a dentin bonding agent or varnish may be applied over the Glu/Sense treated dentin.

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