NoMIX® Temporary Cement for Crowns & Bridges

ENGLISH

INSTRUCTIONS FOR USE

NoMIX® Temporary Cement

Moisture Activated Temporary Cement for Crowns and Bridges
REF 290040, 310157, 310520

DESCRIPTION: NoMIX Temporary Cement is a single-component, moisture activated, non-eugenol, white, temporary cement designed for short-term cementation of crowns and bridges. Intended to be used for less than 30 consecutive days.

INDICATIONS: NoMIX Temporary Cement may be used with crowns made of temporary or permanent materials (acrylic, composite, PFM, and ceramic).

DIRECTIONS:

- Leave prepared tooth moist. Do not dry.
- Lightly cover the inside and margins of the dry crown with NoMIX Temporary Cement.
- Set the crown and check for fit. Have patient bite down.
- Wait 5 minutes. Cement will be set enough to gently remove any excess cement. Rinse out thoroughly with water. If necessary, clean the interproximals by knotting a piece of floss and pulling it through (not up).
- The cement will fully set in approximately 15 minutes. It is recommended to instruct patient to not eat for at least 1 hour.

STORAGE AND USAGE:

- Store at room temperature (5°C - 25°C; 41°F - 77°F).
- Keep dispenser capped between uses to prevent cement from prematurely hardening.

NoMIX Temporary Cement for Crowns and Bridges

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SAFETY DATA SHEET (SDS)

In accordance with Hazard Communication Standard (HCS), 29CFR 1910.1200(g), 2012 revision

SECTION I - PRODUCT / COMPANY IDENTIFICATION

Product Trade Name: NOMIX® Temporary Cement.

Moisture activated temporary cement for crowns and bridges

Part (Item) Numbers: 290040, 310157, 310520

Manufacturer: CENTRIX INC

Address: 770 RIVER ROAD

City, State, Zip: SHELTON, CT 06484

Emergency Telephone Number: (203) 929-5582

Information Telephone Number: (203) 929-5582

Date Revised: 09/2014

SECTION II – HAZARD IDENTIFICATION

Designation of Risk: Xi, irritant ⚠️ ☠️

Special Details of Risk (R-categories): 36-38

Detrimental effects/symptoms: Irritant in case of eye, skin and inhalation contact.

May cause sensitization by skin contact.

SECTION III - COMPOSITION / INFORMATION ON INGREDIENTS

Component	%	CAS#	EINECS #
D.M.T.G (Glycol)	<20%	143-24-8	205-594-7
Ethyl Methacrylate Polymer	<4%		
Calcium Sulfate	<31%	10034-76-1	231-900-3
Zinc Oxide	<26%	1314-13-2	
Triacetin	<3%	102-76-1	
Potassium Sulfate	<21%	7778-80-5	231-915-6

Exact % content withheld due to trade secret restrictions.

SECTION IV - FIRST AID MEASURES

Skin: Wash with soap and water.

Ingestion: N/A Inhalation: N/A

Eyes: Flush with copious amounts of water.

If irritation persists seek medical attention

SECTION V - FIRE-FIGHTING MEASURES

Burning produces irritating, toxic and obnoxious fumes.

Suitable extinguishing media: Use extinguisher suitable to cause of fire and its surroundings.

Forbidden extinguishing media: not available

Special firefighting measures: not available

Unusual fire and explosion hazards: not available

Special protection equipment: Wear suitable respiratory equipment when necessary.

SECTION VI - ACCIDENTAL RELEASE MEASURES

Transfer bulk into suitable containers for disposal. Wipe residue with damp cloth

and scrape up into suitable container for disposal.

Personal precautions: Ensure adequate ventilation of the working area.

Environmental precautions: Do not allow product to enter drains. Prevent further spillage if safe.

Reclaiming methods: Sweep up. Transfer to suitable, labelled containers for disposal.

Clean spillage area thoroughly with plenty of water.

SECTION VII - HANDLING AND STORAGE

Handling: No handling problems are likely to be encountered provided normal safe practices of hygiene are applied.

Storage: Store in a cool dry place.

Containers: Keep containers tightly closed. Store in correctly labelled containers

Incompatibilities: unknown

Other precautions: no further information available

SECTION VIII - EXPOSURE CONTROLS / PERSONAL PROTECTION

General measures: Wash hands thoroughly after contact with this product.

Limits: no data is available

Respiratory protection: Wear suitable respiratory equipment when necessary.

Eye protection: Goggles / eyeshields

Skin protection: Overalls, rubber gloves

Hands protection: Overalls, rubber gloves

Other equipment: no further information available

SECTION IX - PHYSICAL AND CHEMICAL PROPERTIES

Appearance: White Stiff Paste

Explosion limits: N/A

Odor: Odorless

Water solubility: Insoluble

Boiling point: N/A

Evaporation rate: N/A

Flash point N/A

Other Data: Not available or none.

SECTION X - STABILITY AND REACTIVITY.

Stability: Stable, non-reactive

Conditions to avoid: N/A

Hazardous decomposition products: N/A

Hazardous polymerization products: N/A

SECTION XI - TOXICOLOGICAL INFORMATION.

No data is available

SECTION XII - ECOLOGICAL INFORMATION.

No data is available

SECTION XIII - DISPOSAL CONSIDERATIONS.

No data is available

SECTION XIV - TRANSPORT INFORMATION.

No data is available

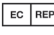
SECTION XV - REGULATORY INFORMATION

No data is available

SECTION XVI - OTHER INFORMATION

Handling of product exclusively by dental specialists according to instructions for use.

CAUTION: The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/procedure will be exclusively under the user’s responsibility.

	
<i>Making Dentistry Easier.SM</i>	
	CENTRIX INCORPORATED 770 River Road, Shelton, CT 06484 USA (203) 929-5582 • (800) 235-5862 • FAX (203) 929-6804 www.centrixdental.com
	Dental-Contact Vertriebs OHG, Im Spiet 101, DE 26506 Norden
	
	EMERGO AUSTRALIA Level 20, Tower II, Darling Park, 201 Sussex Street Sydney, NSW 2000, Australia