# Tempit-E

Moisture-Activated Temporary Filling and Sealing Material with Eugenol

## GB - ENGLISH INSTRUCTIONS FOR USE

**PRODUCT DESCRIPTION:** Tempit-E is a viscous, calcium sulphate paste containing eugenol. The material hardens when moisture-activated by water or saliva and expands slightly when completely set. The addition of zinc oxide and barium sulphate provide radiopacity.

**INDICATIONS:** Short-term temporary fillings, single use only.

**DIRECTIONS:**

- Prepare site to apply Tempit-E filling material; cavity should be slightly moist.
- Remove one tube from the jar and place into a C-R syringe; remove the cap.
- Squeeze the syringe handle with slow, steady pressure, delivering Tempit-E into the cavity site; dispose after single use.
- Tamp down with moist instrument; wait 5 minutes and trim as needed before dismissing patient.
- On patient's return visit, remove Tempit-E with an explorer.

**CAUTIONS & WARNINGS:**

- Tempit-E contains a methacrylate resin; do not apply if patient has allergic potential.
- Tempit-E is a temporary filling material; intended to be used for 30 days or less.
- Store unused Tempit-E in supplied jar at all times to prevent premature hardening of material.
- Dispose of Tempit-E properly according to local and national requirements.
- Do not use Tempit-E beyond expiration date stated on container.

**DISCLAIMER:** Centrix products have been developed for use in dentistry. We have provided written, verbal and online information as to the use of each product. This information does not relieve you from examining our products upon receipt to determine their suitability for use in the intended manner of application. The usage of our products is beyond our control and the user maintains full responsibility. We guarantee the quality of our products in accordance with the existing standards and corresponding to the conditions stated in our instructions for use and safety data sheets.

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## SAFETY DATA SHEET (SDS)

In accordance with Hazard Communication Standard (HCS), 29CFR 1910.1200(g), 2012 revision

**SECTION I – PRODUCT / COMPANY IDENTIFICATION**
**Product Name: Tempit®- E TEMPORARY FILLING MATERIAL**
Moisture activated eugenol temporary filling material for lost or loose caps
Synonyms: Temporary filling material
Part (Item) Number: See Above
Company Name: Centrix, Inc.
Address: 770 River Road, City, State, Zip: Shelton CT, 06484
Emergency Telephone Number: 203-929-5582
Information Telephone Number: 800-235-5862

**SECTION II – HAZARD IDENTIFICATION**

2.1: Classification
2.1.1: Classification according to Regulation (EC) No 1272/2008 [CLP]
Hazard Class: Skin sensitization
Hazard Category: 1
Hazard Statement: H317

2.2: GHS Label Elements. Hazard Pictograms
Signal Word: WARNING
Restricted to use by dental professional only

Hazard Statements
H317: Sensitization. 1. May cause an allergic skin reaction

Precautionary Statements
P261: Avoid breathing vapor
P280: Wear protective gloves and eye protection
P302+P352: IF ON SKIN: Wash with plenty of water
P333+P313: If irritation or rash occurs, get medical advice/attention
P391: Collect spillage

**SECTION III – COMPOSITION / INFORMATION ON INGREDIENTS**

Hazardous Components	%	CAS#	EINECS #
Polyethyl Methacrylate	1-10%	9011-14-7	
Barium Sulfate	10-20%	7727-43-7	
Zinc Oxide	1-10%	1314-13-2	215-222-5
Eugenol	1-10%	97-53-0	202-589-1

Exact % content withheld due to trade secret restrictions.

**SECTION IV – FIRST AID MEASURES**

Skin: Wash with soap and water

Ingestion: May cause nausea and vomiting. Seek medical attention if persists.

DO NOT INDUCE VOMITING

Inhalation: Irritating to respiratory system. May cause coughing, tightness of the chest. Move exposed person to fresh air. Seek medical attention.
Eyes: Flush with copious amounts of water for 15 minutes with eyes wide open. If irritation persists seek medical attention.

**SECTION V – FIRE-FIGHTING MEASURES**
Burning produces irritating, toxic and obnoxious fumes. Suitable extinguishing.
Media: Use extinguisher suitable to cause of fire and its surroundings.
Special protection equipment: Wear suitable respiratory equipment when necessary.

**SECTION VI – ACCIDENTAL RELEASE MEASURES**
Avoid raising dust. Ensure adequate ventilation of the working area. Wear suitable protective equipment.

**SECTION VII – HANDLING AND STORAGE**
Handling: No handling problems are likely to be encountered provided normal safe practices of hygiene are applied.
Storage: Store in a cool dry place
Containers: Keep containers tightly closed. Store in correctly labelled containers.
Incompatibilities: Unknown

**SECTION VIII – EXPOSURE CONTROLS / PERSONAL PROTECTION**
General measures: Wash hands thoroughly after contact with this product.
Limits: No data is available
Respiratory Protection: Wear suitable respiratory equipment when necessary
Eye protection: Goggles / eye shields
Skin protection: Overalls, rubber gloves
Hands protection: Overalls, rubber gloves
Other equipment: No further information available

**SECTION IX – PHYSICAL AND CHEMICAL PROPERTIES**
Appearance: White Stiff Paste
Odor: Odorless
Boiling point: N/A
Flash point: N/A
Other Data: Not available or none
Explosion limits: N/A
Water solubility: Insoluble
Evaporation rate: N/A

**SECTION X – STABILITY AND REACTIVITY**
Stability: Stable, non-reactive
Conditions to avoid: Heat. Moisture
Materials to avoid: N/A

**SECTION XI – TOXICOLOGICAL INFORMATION**
Barium Sulfate: Oral Rat LD50: 15000mg/kg
Eugenol: Oral Rat LD50: 1930mg/kg
Polyethyl methacrylateOral Rat LD50: 5000mg/kg

**SECTION XII – ECOLOGICAL INFORMATION** - No data is available

**SECTION XIII – DISPOSAL CONSIDERATIONS**
Dispose in accordance with local authorities

**SECTION XIV – TRANSPORT INFORMATION** - Not classified as dangerous goods

**SECTION XV – REGULATORY INFORMATION** - No data is available

**SECTION XVI – OTHER INFORMATION** - None

**CAUTION:** The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/procedure will be exclusively under the user’s responsibility.

<b>centrix®</b> MAKING DENTISTRY EASIER	
	<b>CENTRIX INCORPORATED</b> 770 River Road, Shelton, CT 06484 USA (203) 929-5582 (800) 235-5862 FAX (203) 929-6804 www.centrixdental.com
 <b>0482</b>	
 	Dental-Contact Vertriebs OHG, Im Spieß 101, DE 26506 Norden
 	<b>EMERGO AUSTRALIA</b> Level 20, Tower II, Darling Park, 201 Sussex Street Sydney, NSW 2000, Australia

©2022 Centrix, Inc. | Made in the USA | Date Revised: 3/2022 | 001108 REV B