



**1** Select appropriate size GingiCap™.



**2** Attach the mixing nozzle to the GingiTrac Retraction MiniMix.




**3** Load MiniMix into Snub Nose Snap-Fit™ Syringe.



**4** Express a “pea-sized” amount and discard. This is to ensure that the catalyst and base are being mixed thoroughly.



**5** Syringe the GingiTrac Retraction material directly around and over the tooth preparations' margin.

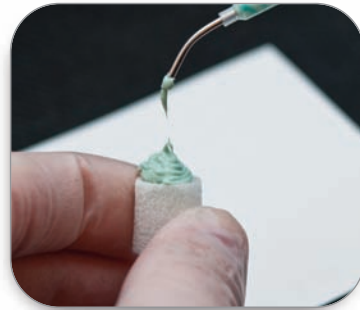

**6** Fill the GingiCap with GingiTrac Retraction material and place back over the preparation.



**7** Have patient close with firm but not excessive pressure. Wait 3 to 5 minutes, depending on the amount of bleeding or oozing for complete set and retraction.



**8** Ready for impression.

## HELPFUL HINTS:

- Do not use the set GingiTrac as a final impression.
- MiniMixes require the use of any Centrix' C-R<sup>®</sup> Syringe.
- Use vinyl gloves. Do not handle material with latex or rubber gloves.

