



## An easy to use non-eugenol cement with calcium hydroxide

*Softer, semi-elastic set allows easier removal and minimal cleanup*

Access Temporary Cement is the perfect choice for short-term cementation of crowns and bridges. The auto mix system eliminates pad mixing and delivers a perfect mix every time. And the small hand syringe eliminates the need for a bulky mixing gun. A softer, semi-elastic set allows for easier removal and minimal clean up.

Access is an eugenol-free temporary cement with calcium hydroxide; is easy to process; and does not influence the setting of resin-based filling and luting materials. Access is recommended for patients with known allergies to eugenol. Access Temporary Cement has thick and semi-elastic properties for safe adhesion to a temporary crown or bridge. Flexible set for good retention and easy removal.

Containing calcium hydroxide, Access Temporary Cement supports tooth vitality with the formation of secondary dentine and provides a bacteriostatic effect. Access Temporary Cement is packaged in a single 5ml automix cartridge that allows for no mess, bubble-free mixing; fast and direct application and the smallest automix tips on the market.

Working time  $\geq$  2 minutes

Setting Time (37°C) = 4 – 10 minutes

Film thickness  $\leq$  24  $\mu$ m

Store Access at temperatures between 39 - 73°F (4 - 23°C).  
Close tubes tightly after use.

Ordering is easy. Call 800-235-5862 or contact your preferred dealer.  
Visit [centrixdental.com](http://centrixdental.com) for more information.

CE  
0510

CENTRIX  
**Access**<sup>®</sup>  
Automix Temporary Cement



**centrix**<sup>®</sup>  
Making Dentistry Easier.<sup>SM</sup>

## Instructions for Use



1. Remove the cap from the auto mix syringe, bleed a "pea-sized" amount from the syringe onto a mixing pad.
2. Place and lock the mixing nozzle type 10 with a 90° turn clockwise.
3. Apply a thin layer of Access inside the temporary crown or bridge. Access Temporary Cement remains workable for about 1 minute. All materials set in the mouth within 4 - 6 min.
4. Place the temporary onto the prepared tooth.
5. Remove any material excess after 3 - 5 min.

### Indications:

Access is used for cementing temporary crowns and bridges, inlays and on-lays.

### Contraindications:

Access contains zinc oxide, magnesium oxide, calcium hydroxide, fatty acids, modified resin and vegetable oils. Do not use in cases of known sensitivities or allergies.



### ORDERING INFORMATION:

**ACCESS<sup>®</sup> TEMPORARY CEMENT**  
 For Crown & Bridges

REF 310521 (1) x 5ml auto mix cartridge + 10 nozzles



**Our Guarantee** — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

**Ordering is easy.**  
 Call 1-800-235-5862  
 or contact your preferred dealer.

[centrixdental.com](http://centrixdental.com)



**centrix**<sup>®</sup>  
 Making Dentistry Easier.<sup>SM</sup>