



When first impressions count

You can rely on Access super hydrophilicity impression materials for consistently accurate impressions. Featuring several precise viscosities that meet strict ADA guidelines, viscosities and set times to satisfy every technique. Access vinyl polysiloxanes feature low contact angles to work accurately in a wet environment reducing the chance of costly, time-consuming re-takes.

Choose from our complete line of ADA-standard viscosities:

- Fast setting Access Blue and regular setting Access Purple feature a mousse-like viscosity for bite techniques.
- Access Blue sets hard like laboratory plaster in 30 seconds when taking bite registrations, provisional crown pre-op impressions, and so many other "quick" techniques. Blue allows you to cut and trim the impression without it crumbling.
- Low viscosity Access Gold and medium viscosity Access Monophase™ are syringe-able through Access needle tips.
- Both Access Gold and Access Monophase feature hydrophilic additives for high accuracy and low contact angles.

Ordering is easy. Call 800-235-5862 or contact your preferred dealer.
Visit centrixdental.com for more information.



CENTRIX
Access® VPS
Vinyl Polysiloxane for All Impression Techniques



centrix®
Making Dentistry Easier.™

Access Impression Materials Setting Times

PRODUCT	VISCOSITY	TECHNIQUE	MIXING TIME	WORK TIME	SET TIME IN MOUTH	TOTAL SET TIME
Gold	Low	Syringe Injection	Auto	2:30	1:00	2:30
Monophase	Medium	One Step	Auto	2:30	1:00	2:30
Heavy Body	High	Tray	Auto	2:30	1:00	2:30
Putty	Putty	Tray	:30	2:00	2:00	4:00

Access Bite Registration Material Setting Times

PRODUCT	VISCOSITY	TECHNIQUE	MIXING TIME	WORK TIME	SET TIME IN MOUTH	TOTAL SET TIME
Blue	Fast Bite	Bite Registration	Auto	:30	1:00	1:30
Purple	Bite	3.N.One Tray	Auto	2:30	1:00	3:30



ACCESS IMPRESSION MATERIALS

Kits include 2 x 50ml, S-50 cartridges plus 6 mixing nozzles

- REF 360001 Access Purple®
- REF 360002 Access Gold®
- REF 360006 Access Blue®
- REF 360016 Access Heavy™
- REF 360017 Access Monophase™
- REF 360015 Access Putty™ (2 x 280ml jars)



SUGGESTED ACCESSORY:

3.N.ONE™ IMPRESSION TRAYS

For the Double-Bite Impression Technique

- REF 360011 35 Anterior Trays
- REF 360012 50 Posterior Trays
- REF 360013 35 Quadrant Trays
- REF 360014 40 Assorted Trays

- REF 360005 Access Tray Adhesive (10 ml)
- REF 360036 DS50 1:1 Automix Syringe
- REF 360010 DS50 Mixing Nozzles, 24 (Teal hub)



Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Ordering is easy.
Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com



centrix®

Making Dentistry Easier.™