

## Do your spiral paste fillers create voids when placing calcium hydroxide?



*Calciject's single, unit-dose tip provides direct delivery into the canal and fills from the bottom up.*

Our unique delivery system allows you to inject Calciject without having to worry about voids as you twist out the spiral paste filler. Our system allows for easier cleanup and no cross contamination because you simply throw out the cartridge after use.

Calciject was developed specifically for medicating the canal between endodontic treatments. Simply bend the tip as needed and inject the paste. Through the needle tip, the paste fills the canal from the bottom up, treating the whole canal. Throw out the tip and you are ready to place a sealing material, such as Tempit.<sup>®</sup> At Centrix we make dentistry easier.

- No mixing, no mess
- Single-dose syringe tips for easy placement, maximum asepsis
- Bendable 20ga NeedleTube syringe tip
- Superior results compared to spiral paste fillers
- Radiopaque
- Works in any C-R<sup>®</sup> syringe



Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit [centrixdental.com](http://centrixdental.com) for more information.



**CENTRIX**  
**Calciject**<sup>®</sup>

Calcium Hydroxide Paste  
for Root Canal Treatment



**centrix**<sup>®</sup>

Making Dentistry Easier.<sup>SM</sup>

# CENTRIX Calciject®


Calcium Hydroxide Paste  
for Root Canal Treatment

Centrix



Placing material with C-R® Tubes and Plugs creates void- and porosity-free restorations.

The competition

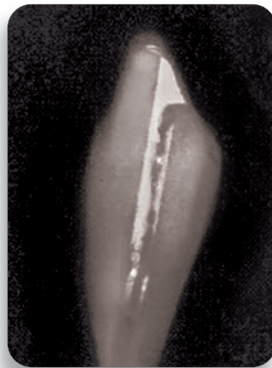

Placing material traditionally with spatulas or spiral paste fillers.

## INSTRUCTIONS FOR USE



1) Remove Calciject cartridge from package.



2) Insert into Centrix C-R syringe and pull off sheath cover.



3) Bend needle as needed. Test first on pad.



4) Squeeze syringe with a slow, steady pressure and inject into prep.



5) Easy cleanup.

## ORDERING INFORMATION:

Calciject  
REF 320003      24 x .20ml Calciject tips

### Suggested Accessory

REF 130650      Snap-Fit™ Syringe



**Our Guarantee** — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

**Ordering is easy.**  
Call 1-800-235-5862  
or contact your preferred dealer.

[centrixdental.com](http://centrixdental.com)



**centrix**®

*Making Dentistry Easier.™*