

CENTRIX  
**Encore<sup>®</sup> D/C**  
**MiniMix<sup>®</sup>**  
Unit Dose Dual-Cure Core Buildup Material

## Makes Core Buildup Easier

The MiniMix unit-dose technology eliminates waste and cross-contamination concerns.



### Single-dose. Great price. Problems solved.

Encore<sup>®</sup> D/C MiniMix<sup>®</sup>: Our most popular format, the MiniMix .5mL unit-dose technology eliminates waste and cross-contamination concerns. The smaller size is more easily maneuvered in the mouth, while the longer mixing tip enables easy application down canals in second and third molars. MiniMix is also patient-friendly; the small format is less threatening and more comfortable for sensitive patients.

#### Features:

- Superior core buildup material that ensures completely cured materials, structural integrity plus ease of use
- Light activate for 20 seconds to accelerate the set
- NeedleTube<sup>®</sup> delivers material to bottom of the post hole
- Non-slumping formula requires no matrix or core form
- Cuts like dentin to reduce ditching
- Radiopaque
- Contains fluoride



**Encore<sup>®</sup> D/C MiniMix<sup>®</sup>**  
**is the only dual-cure**  
**sold by dose.**

**6 -steps for building a void-free, bonded, composite post and core:**




1. Enlarge root canal to accept Encore D/C needletip. Bevel the edges to eliminate sharp angles.




2. Etch the post hole, rinse thoroughly and dry.




3. Apply bonding agent according to manufacturer's instructions.

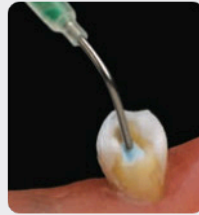

4. Insert needletip to the bottom of the post hole and inject material with slow steady pressure.

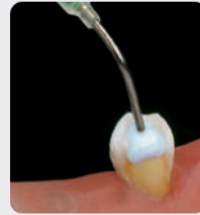

5. Continue filling by injecting and slowly withdrawing the needletip.

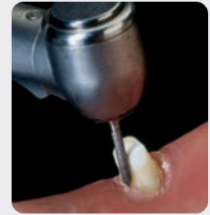

6. If desired, light-activate Encore D/C to accelerate the set. Allow material to fully cure and then cut and shape the core. The result is a solid, securely bonded post and core.



**Encore D/C MiniMix Specifications**

Comprehensive Strength	222 MPa
Flexural Strength	110 MPa
Diametral Tensile Strength	48 MPa
Light-activated Depth of Cure	2mm 40 Seconds
Shrinkage	1.4%

**ORDERING INFORMATION**

**Encore D/C MiniMix**

Contains 36 single-dose cartridges (0.5mL net ea.) and 72 mixing nozzles.

REF 310254 Natural Kit 36 pack  
 REF 310129 Natural Kit 12 pack



REF 310255 Contrast Blue 36 pack  
 REF 310130 Contrast Blue 12 pack



**Suggested Accessories**

REF 130651 Snub Nose Snap-Fit Gun  
 REF 910630 MiniMix Nozzles, Clear Hub, 10-pack

Ordering is easy. Call **800.235.5862** or contact your preferred dealer.  
 Visit **centrixdental.com** for more information.

