

CENTRIX GingiTrac™ MiniMix®

Hands-Free Gingival Retraction System



No big bulky automix gun.
And still no cleanup or
packing cord.

Hate the big automix gun? GingiTrac™ MiniMix®, our newly re-formulated 1:1 astringent, Vinyl Polysiloxane formula is now in our patented, single-dose MiniMix.

More patient-friendly, the MiniMix is easier to express than the automix gun with less waste. Plus, the smaller size and ease of use will save valuable chair time and reduce patient stress.

It's easy: Select appropriate GingiCap size. Syringe the retraction paste into the GingiCap and around the prep. Bite down. Wait 2 minutes. MiniMix technology virtually eliminates waste, and allows for easy set up and no cleanup.

Still the only retraction material and technique that lets you retract the gingiva on up to six preps with one application with no packing cord or cleanup. GingiTrac's built-in astringent controls oozing, while the flowable VPS material gently pushes the gingiva.

- Works in 2 minutes, regardless of number of preps
- Get more accurate impressions
- No hands in mouth
- Works gently, no tissue trauma or ligament damage
- Contains aluminum sulfate astringent to control bleeding and oozing
- Removes cleanly, in one piece, without rinsing



GingiTrac used with GingiCap for single prep retraction.



Single-Unit Retraction Technique:



Select appropriate GingiCap™ size.



Attach the mixing nozzle to the GingiTrac Retraction MiniMix.



Load MiniMix into Snub Nose Snap-Fit™ Syringe



Express a “pea-sized” amount and discard. This is to ensure that the catalyst and base are being mixed thoroughly.



Syringe the GingiTrac Retraction material directly around and over the tooth preparations' margin.



Fill the GingiCap with GingiTrac Retraction material and place back over the preparation.



Have patient close with firm but not excessive pressure. Wait 2 minutes, depending on the amount of bleeding or oozing for complete set and retraction.



Bite down and wait 2 minutes.

ORDERING INFORMATION

GINGITRAC MINIMIX STARTER KIT

REF 360068 24 Cartridges
 24 Mixing nozzles
 20 Regular-sized GingiCaps
 20 Large-sized GingiCaps
 1 Snub-Nose Snap-Fit Syringe

GINGITRAC MINIMIX REFILL KIT

REF 360069 48 Cartridges
 48 Mixing nozzles
 40 Regular-sized GingiCaps
 40 Large-sized GingiCaps



SUGGESTED ACCESSORIES

REF 130651 Snub Nose Snap-Fit Syringe
 REF 910630 MiniMix nozzles, 10

Ordering is easy. Call 800.235.5862 or contact your preferred dealer.
 Visit centrixdental.com for more information.

Snap-Fit, GingiTrac and GingiCap are trademarks of Centrix, Inc. Minimix is a registered trademark of Centrix, Inc.
 ©2012 Centrix, Inc. All rights reserved. MK0052 REV B



centrix®

Making Dentistry Easier.™