

# CENTRIX GingiTrac™

Hands-Free Gingival Retraction System



The first vinyl polysiloxane retraction system that does it all: retraction, hemostasis, comfort. **Guaranteed.**

## VPS, not clay.

GingiTrac is a unique 1:1 vinyl polysiloxane retraction material that gently retracts the gingiva, controls bleeding and fluid flow, and is more comfortable for your patients. And because it's VPS, cleanup is easier – simply rinse it away – no break up, no residual debris, no mess.

## Gentle tissue retraction

Simply syringe GingiTrac into the sulcus and around the tooth with our patented NeedleTip™, apply bite pressure with GingiCap™, and wait two minutes. It's that easy to achieve full retraction with complete exposure of all margins.

## Controls bleeding and fluids

GingiTrac's built-in astringent (15% aluminum sulfate) quickly constricts capillaries to stop bleeding. VPS + bite pressure displaces unwanted crevicular fluids. The result – a dry sulcus for bubble-free margins.

## Better for your patients

No packing cord. No damage to epithelium or periodontal ligament. No inadvertent tissue tears. GingiTrac syringes easily around teeth making access easy and placement more comfortable with no post-operative pain.



Unlike traumatic cord techniques or messy paste alternatives, our patented GingiTrac material delivers the perfect combination of built-in astringency, gentle retraction and fast setting times to assure the most accurate impressions.



**centrix**®

Making Dentistry Easier.™

## Single unit technique:



Select appropriate GingiCap™ size.



Dispense GingiTrac into the GingiCap.



Dispense GingiTrac around the prep.




Bite down and wait two minutes.



Fully exposed, dry margins. Ready for impression.

## Multi-unit technique:

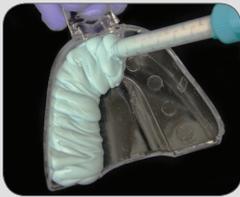

Express Access BLUE® into a rigid plastic tray. Impress.



Dispense GingiTrac into the Access BLUE impression.



Dispense GingiTrac around the prep.



Bite down and wait two minutes.



Fully exposed, dry margins on all units. Ready for impression.

## Earn More with Every Kit

When paired with GingiTrac material, our NEW SuperMixer nozzles reduce waste, saves time and increases profits for your practice.

Waste Reduction (avg.) <sup>1</sup>	Additional Uses (avg.) <sup>2</sup>
33%	10

## Ordering Information:

### GingiTrac Refill Kit

**REF 360100** 2 GingiTrac cartridges, 10 SuperMixer mixing nozzles and 10 Dispensing tips

### GingiTrac Standard

**REF 360101** 1 Mixing Gun, 2 GingiTrac cartridges, 40 SuperMixer mixing nozzles, 40 Dispensing tips, 30 Regular GingiCaps and 30 Large GingiCaps

### GingiTrac Plus

**REF 360102** 4 GingiTrac cartridges, 80 SuperMixer mixing nozzles, 80 Dispensing tips, 60 Regular GingiCaps and 60 Large GingiCaps

### GingiCap Regular-Size Refill Kit

**REF 360103** 40 Regular GingiCaps, 40 SuperMixer mixing nozzles and 40 Dispensing tips

### GingiCap Large-Size Refill Kit

**REF 360104** 40 Large GingiCaps, 40 SuperMixer mixing nozzles and 40 Dispensing tips

### Suggested Accessories:

**REF 360006** Access BLUE Impression Material

**REF 910632** SuperMixer Pink Hub, 50

**REF 290017** Nozzle Tips, 144, 18ga Yellow

**REF 360036** 50mL 1:1/2:1 Automix Gun



**Ordering is easy. Call 800.235.5862 or contact your preferred dealer. Visit [centrixdental.com](http://centrixdental.com) for more information.**

1. Average waste reduction vs. old-style nozzles, per 50mL cartridge using Centrix restoratives. Results will vary depending on material.

2. Average number of additional core build-ups per 50mL cartridge