

CENTRIX
LubeCube[®]
Composite Instrument Cleaner and Lubricator


**Cleans, conditions
and lubricates instruments.**



Introducing the LubeCube[®]. Clean and lubricate your sculpting instrument by simply running it through the slit on top.

Put an end to composite stickiness, pull-back and unnecessary voids. With LubeCube, simply peel off the adhesive back, stick it to a surface and slide your instrument tip through the slit. The cleaning and lubricating action will allow you to shape with ease. At Centrix, we make dentistry easier.

- Cleans, conditions and lubricates instruments with an unfilled, light-activated, Bis-GMA/UDMA resin monomer
- Lubricant is compatible with all light-activated and dual-cure composites. No alcohol, no solvents.
- Self-adhesive back allows placement anywhere that is convenient
- No-drip dispensing of lubricant micro-coats each instrument
- Sculpt composites faster and easier, with less "pull-back"
- Makes any composite instrument "non-stick"
- Opaque foam cube keeps resin lubricant from polymerizing
- Low cost, single use and disposable
- Smoother, less porous composite surfaces (see illustration on reverse side)

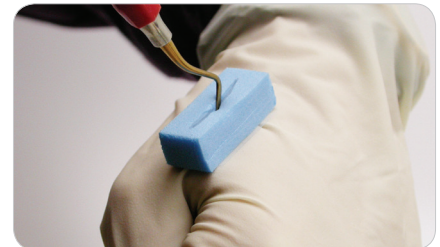


The LubeCube cleans and lubricates your sculpting instrument to make shaping composites faster and easier.




centrix[®]

Making Dentistry Easier.SM


LubeCube Instructions

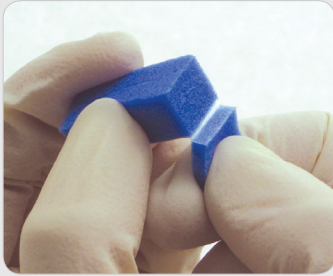


STEP 1 Peel backing off bottom of LubeCube.

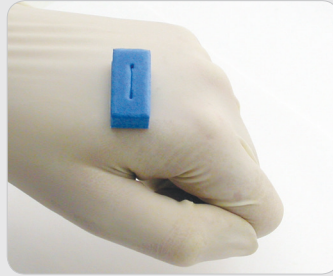

STEP 2 Place and/or stick LubeCube in desired location (back of hand, bib, tray, etc).

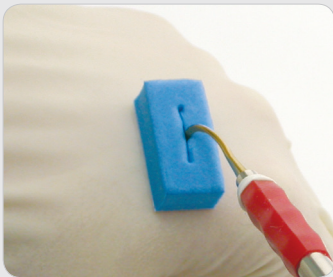

STEP 3 Insert tip of composite instrument into slit of LubeCube. Glide tip of composite instrument back and forth in slit to clean and lubricate.

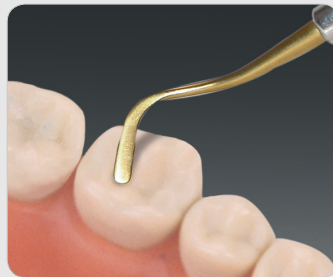


STEP 4 Remove composite instrument and begin sculpting the restoration. Repeat steps one through four as desired until restoration is complete. Dispose of used LubeCube.

SMOOTHER, LESS POROUS SURFACES!

With LubeCube



Without LubeCube

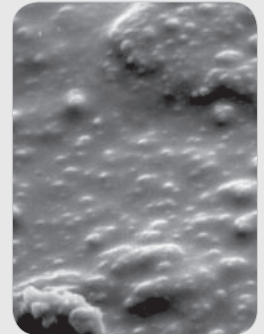

Light-activated 750x magnification

ORDERING INFORMATION

LUBECUBE

Composite instrument cleaner and lubricator.

REF 310108 72 LubeCubes



Ordering is easy. Call 800.235.5862 or contact your preferred dealer.
Visit centrixdental.com for more information.



centrix[®]
Making Dentistry Easier.SM