

NoMIX[®]

Temporary Cement

Single component.
Moisture activated.
Direct dispensing.



NoMIX[®] Temporary Cement is easier for you...and your patients.

NoMIX is the only pre-mixed, single component, temporary cement available. It eliminates the mixing, spatulation and messy cleanup of traditional temporary cements. Yet NoMIX provides superior retention. Pre-loaded in syringes enables fast, exact placement onto restorations. And our new NoMIX take-home applicators are perfect for patient use – at home or on-the-go; they help eliminate those late night emergency calls.



FEATURES:

- NoMIX, no mess, no waste
- Easy to use, in the office or at home
- Non-eugenol, so it's compatible with all resin materials or bonding agents


BENEFITS:

- **Moisture-activated formula** — no need to dry the tooth; reduces post-operative sensitivity
- **Prefilled 1 mL syringe** — perfect for easy, direct application onto restorations
- **Unit-dose, take-home applicators** — take-home security; helps eliminate those late-night emergency calls
- **No eugenol** — won't inhibit resin materials from setting

ACTIVE INGREDIENTS:

- **Potassium Sulfate** — desensitizing effect on the tooth
- **Zinc Oxide** — anti-bacterial ingredient, prevents the “black fungus” and corresponding bad breath
- **Calcium Sulfate** — expands slightly for minimal leakage

Working time 5 minutes. Final set time 15 minutes.

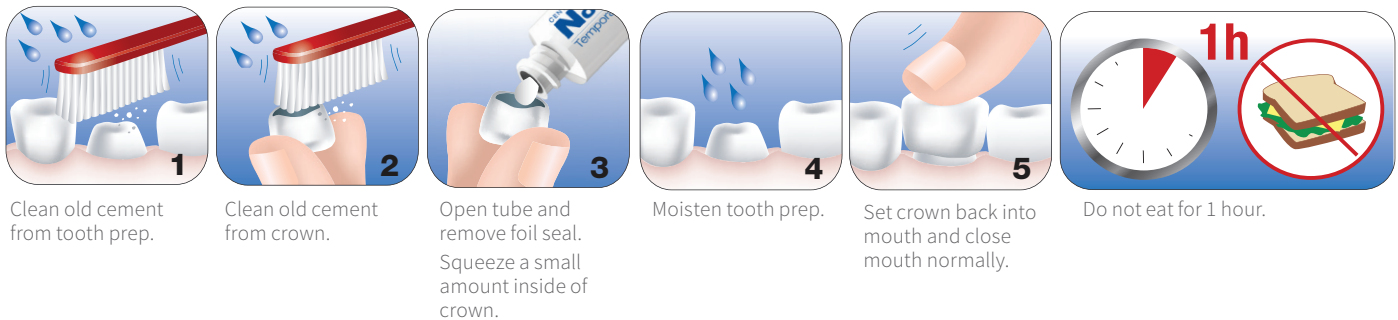

NOMIX TECHNIQUE:

- Leave prepared tooth moist. Do not dry.
- Lightly cover the internal surfaces of the dry restoration with NoMIX Temporary Cement.
- Place restoration and check for fit. Have patient bite down.
- Wait 5 minutes. Cement will be set enough to gently remove any excess cement. Clean the interproximals by knotting a piece of floss and pulling through (NOT up).
- The cement will fully set in 15 minutes. Instruct the patient not to eat for at least 1 hour.

ORDERING INFORMATION:

REF 310157 NoMIX Take-home Applicators

50 x 0.5g unit-dose take-home applicators

REF 310520 NoMIX Starter Kit

6 x 1mL prefilled syringes for use in-office
30 syringe tips
24 x 0.5g unit-dose take-home applicators

REF 290040 NoMIX Syringe Refill Kit

12 x 1mL prefilled syringes for use in-office
60 syringe tips

REF 640002 NoMIX Syringe Tips

30 syringe tips

**Ordering is easy. Call 800.235.5862. Contact your preferred dealer.
Visit centrixdental.com for more information.**